

F Notice de montage **GB** Installation instructions
E Manual de instalación **NL** Montage-instructies
D Montageanleitung



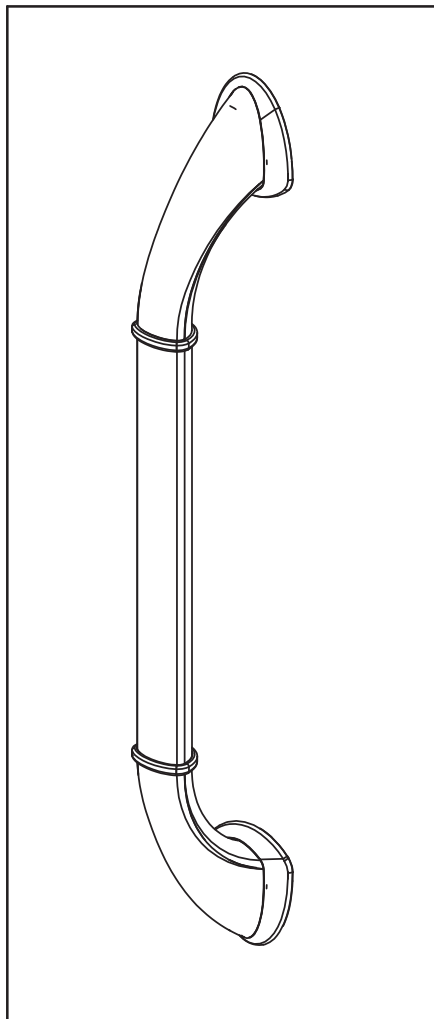
F **IMPORTANT**
GB **IMPORTANT**
E **IMPORTANTE**
NL **BELANGRIJK**
D **WICHTIG**

- F** La colle a une durée de validité, merci de regarder la page suivante.
- GB** This adhesive has a shelf life. Please see the following page.
- E** El pegamento tiene un tiempo de vida limitado, por favor, leer la información detallada.
- NL** De lijm is beperkt houdbaar (zie de volgende bladzijde).
- D** Der Klebstoff ist begrenzt haltbar, siehe hierzu Informationen auf der folgenden Seite.

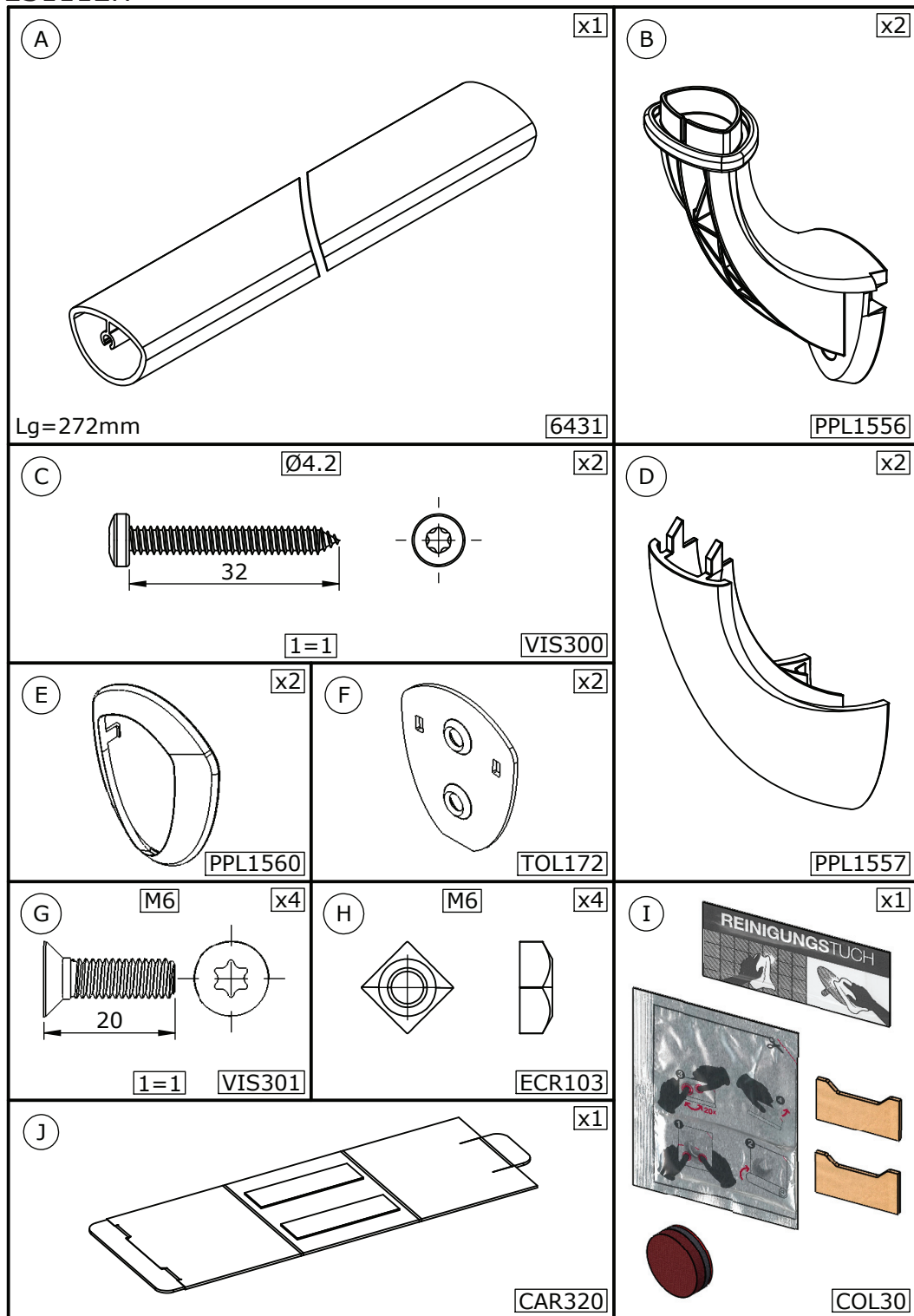● F Matériel nécessaire ● GB Tools required
● E Material necesario ● I Attrezzatura necessaria
● P Nodig gereedschap

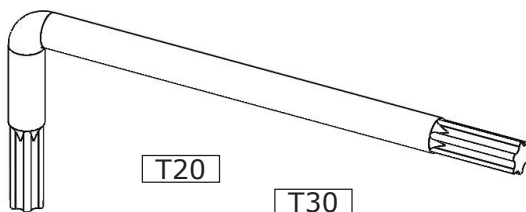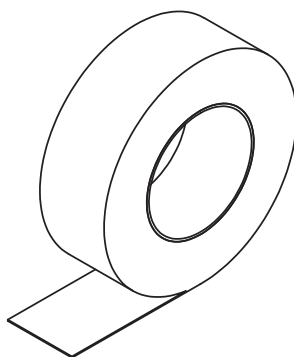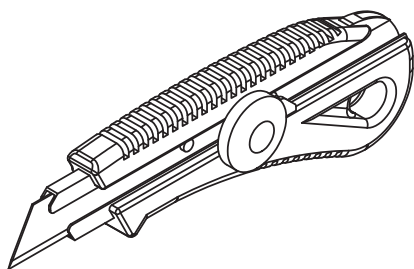F

IMPORTANT

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
GB

IMPORTANT

- On the side of the tube you should see a date of manufacture. The adhesive is suitable for use for **24 months** from this date. Explanations :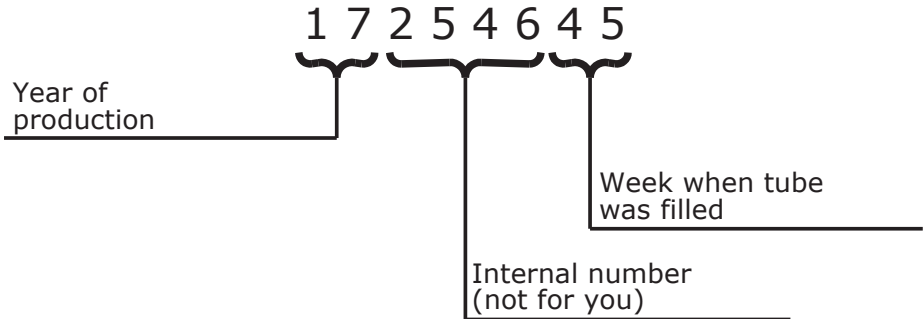

Example above :

Production date : Week 45 of 2017 (November 2017)

So expiry date : Week 45 of 2019 (November 2019)

● Tips for using the glue :

Operating temperature between +5°C et +35°C.
Do not immerse in water for prolonged periods.

● Safety tips :

The contact with skin glue is harmless. Simply rinse with water and soap. In case of contact with eyes, rinse several minutes under water, holding the eyelids open. If swallowed, seek medical advice immediately. Any cured product left over can be disposed of with household waste only in small amounts.

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GB  **Do not install this equipment before reading the following instructions :** **IT**

Fitting instructions. Advised height 80 cm from the ground. Straight grab bars can be installed obliquely (135°) or horizontally.  **Care and maintenance.** IMPORTANT : In order to guarantee the lasting quality of this product and its safety for the user , we recommend that when cleaning it you should use mild soap solutions. CAUTION : DO NOT USE cleaning products made of : etones, hydrocarbonrs, acids, bases, esters and ether. The disinfection has to be made according to the recommendations of use of the various products of disinfection. After use of a cleaning product, make a rinsing in the clear water.  **Safety**

warnings. Grab bars add security to the user if they are fitted according to instructions. Grab bars are not designed to support the total weight of the user, use it for assistance ONLY. DO NOT use grab bar as a step to reach area, do not sit or hang on grab bar. **Use it for assistance ONLY.** The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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